INSTRUCTIONS FOR USE OF:

IM3[®] HAND INSTRUMENTS

FOR VETERINARY DENTAL USE

ENGLISH MANUAL

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ČESKÝ MANUÁL

DANSK MANUAL

SVENSK HANDBOK

Version 1 / 2024

www.im3vet.com


CONTENTS

1. INSTRUMENT USAGE 3

- » iM3 Winged Elevators 3
- » iM3 Luxator 3

2. INSTRUMENT SHARPENING 4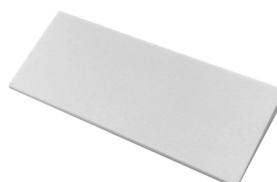

3. TROUBLE SHOOTING & MAINTENANCE 6

- » Autoclave Instructions 6
- » Instrument Staining 6


4. TECHNICAL PRODUCT DETAILS 7

- » Warranty Information 7
- » Technical & Support 7

For more information, please contact customer service in your country :

EU & UK: support@im3dental.com | US: info@im3usa.com | AU: sales@im3vet.com

or your local distributor.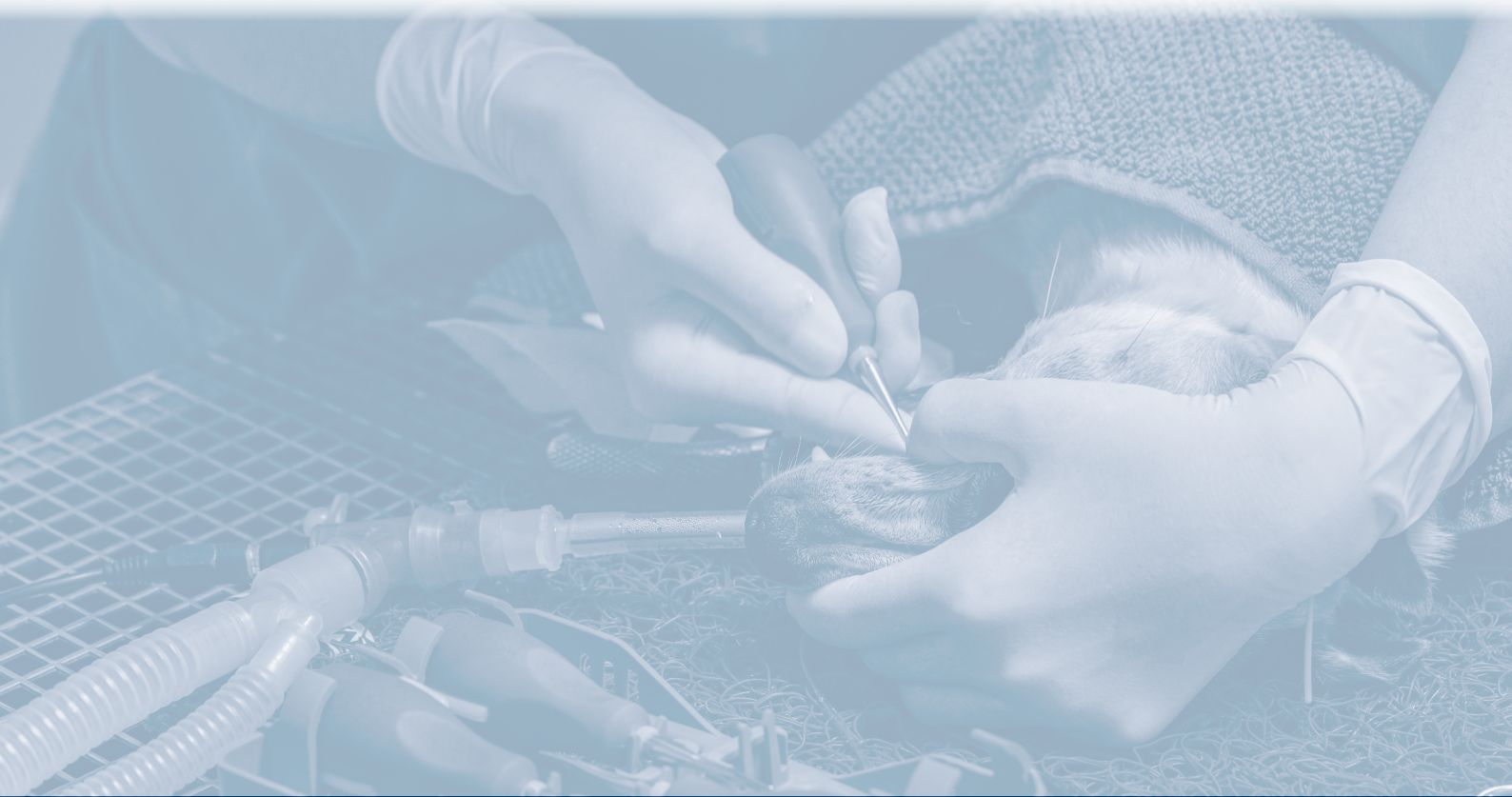

1. INSTRUMENT USAGE

iM3 WINGED ELEVATORS


Warning. DO NOT use these winged elevators with a lever action.

The primary function of winged elevators is to sever the periodontal ligament and expand the alveolus (Tooth socket).

We recommend the elevator tip is moved down the tooth root in a rolling fashion. The winged elevator is then rotated (the wings provide extra leverage) and pressure applied for approximately 10-20 seconds to fatigue the periodontal ligament, causing haemorrhaging to occur inside the periodontal pocket. The haemorrhaging will apply additional pressure to the ligament and aid in the loosening of the tooth. Like any instrument, winged elevators can become dull and require regular sharpening.


Luxators and elevators should be sharpened daily using the iM3 wedge dental sharpening stone, available from iM3 or any of our distributors (iM3 order code: A3110).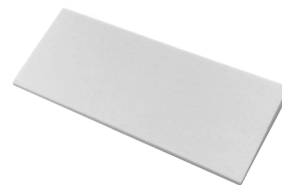

iM3 Wedge Dental
Sharpening Stone
iM3 Code A3110

iM3 LUXATOR

iM3 Luxators, sizes 1-5mm are very fine and sharp. The 5 sizes allow their use for both feline and Canine teeth.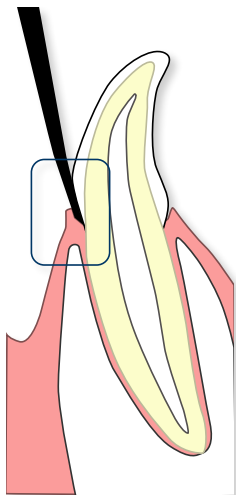


Luxators are used for cutting the periodontal ligament and expanding the alveolus (tooth socket) by inserting the instrument tip into the periodontal space with a gentle side to side rocking motion continuing down the length of the root.

Luxator tips are extremely fine and sharp and can be easily damaged if used as an elevator. The tip is not designed for the extra force used with elevation.

iM3 recommends winged elevators for elevating a tooth to be extracted.

Luxators and elevators should be sharpened daily using the iM3 wedge dental sharpening stone, available from iM3 or any of our distributors (iM3 order code: A3110).

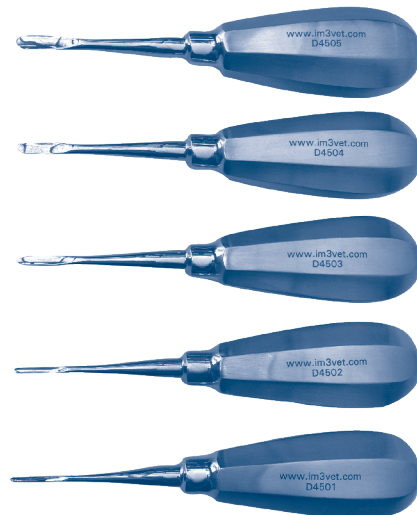Luxator Set with Stubby Handles, Sizes 1 - 5 mm
iM3 Code D4500

2. INSTRUMENT SHARPENING

iM3® WEDGE DENTAL SHARPENING STONE

Cat. Code:
A3110

Sharpening Stone Maintenance

Instruments should be kept sharp and true to their original design. Dental procedures are most effective when using sharp instruments because they reduce fatigue, improve deposit removal, save time and improve tactile sensitivity.

Repeated use of an instrument wears away minute particles of metal from the blade causing the cutting edge to take on a rounded shape resulting in a dull, ineffective blade.

When a blade is dull, the clinician loses the ability to 'feel' the sharp edge 'grabbing' onto a surface. It may then seem that the blade is 'sliding' over the surface or deposit, which causes the clinician to work harder to remove deposits.

For best results, instruments should be sharpened lightly after each use.

STONE CARE

- » After each use, stones should be wiped with a clean cloth to remove metal particles.
- » Alternate the areas used for sharpening to prevent 'grooving' in the stone.

STONE LUBRICATION

- » Stones may be lubricated with oil (iM3 **MD-30**), water or used dry.

Sharpening - STAINLESS STEEL Elevators and Luxators

1. These are sharpened on the flat surface of the stone.
2. The elevator is held in the hand with the index finger just behind the tip.
3. Look for the cutting surface on the winged elevator end and place it flat against the stone at the same angle (indicated below). In the case of the winged elevators pictured, the angle is about 45 degrees.

The picture shows the angle of the instrument against the sharpening stone.


The winged elevator tip is placed flat against the sharpening stone

- » Hold the elevator still in position against the stone and maintaining the angle
- » Move the stone upward, repeating the process 2-3 times. This will sharpen the cutting edge of the instrument.
- » The same process applies to luxators and straight elevators.


Sharpening - iM3 ERGO Elevators and Luxators


Drip a small amount of oil (**MD30**) onto the flat surface of your stone in the shape of a smile or a shallow "U" shape.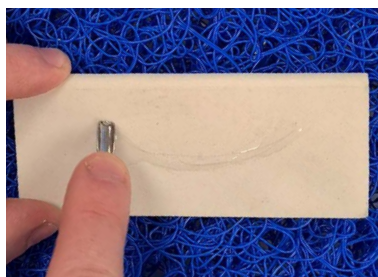

Take the back side of the instrument, the convex surface and place onto the stone. Raise the angle of the instrument up to approximately 45 degrees, so the back of the tip of the blade touches the surface of the stone.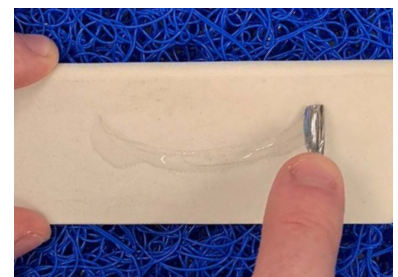

Draw the instrument from right to left in the shape of the "U" back and forth, pressing hard enough to see metal filings on the stone.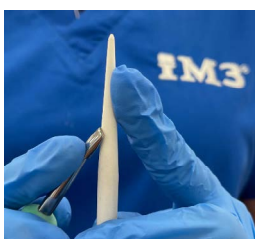

Honing Elevators and Luxators


After sharpening your elevators and luxators it is good practice to also hone the blade using our new sharpening cones (**A3330**) to remove any protrusions or lips on the instrument.

Work the sharpening cone between the wings of the elevator in an upwards motion.


The tapered cone caters for all sizes of luxators and elevators.

iM3[®] SHARPENING GUIDE FOR PERIO INSTRUMENTS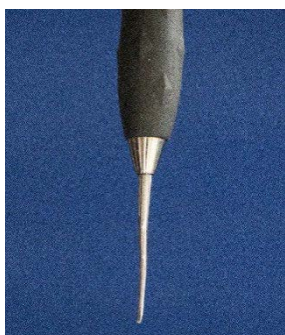


Gracey Curette – Has two opposite 70° angled faces with a curved toe for subgingival scaling

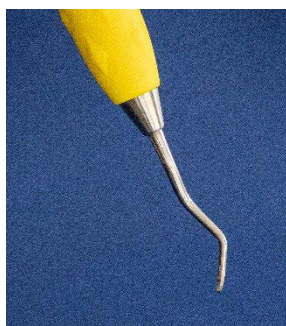Universal Curette – Has a 90° angled face with a curved toe for subgingival scaling

Universal Scaler – Has three 90° angled faces with a pointed tip for subgingival scaling

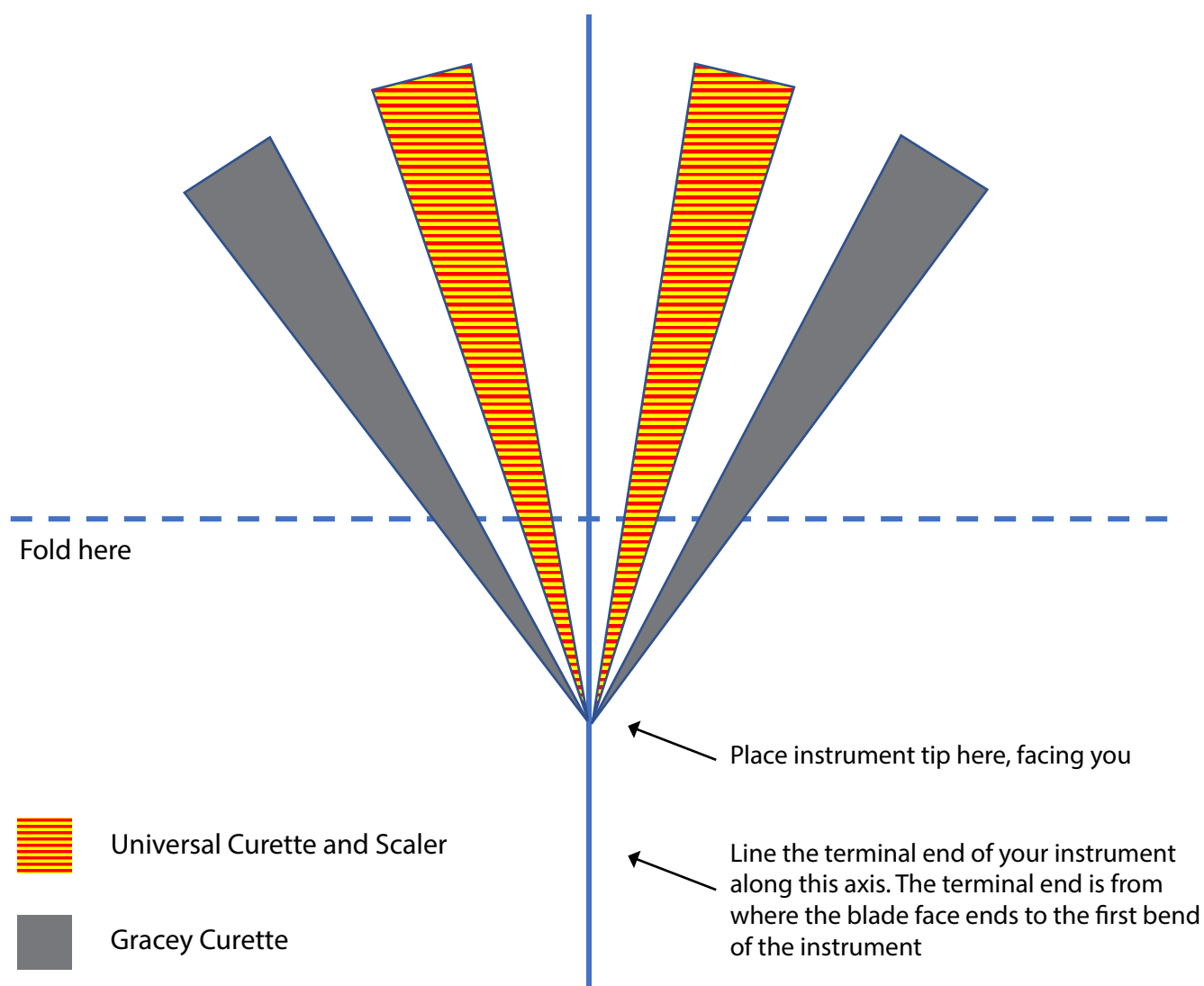Design by Tara Evans

3. TROUBLE SHOOTING & MAINTENANCE

AUTOCLAVE INSTRUCTIONS

Autoclave all plastic at 121°C for 20 minutes, 15psi/1 atm.
(maximum 135°C)

DO NOT use a dry cycle on iM3 Ergo or other instruments with plastic elements as this may cause deformation or weakening of the plastic.

INSTRUMENT STAINING

Staining is a surface deposit on instruments, and most often mistaken for rust. After autoclaving, you may notice a stain on your instruments. Rusting instruments are very rare. Stains on instruments appear in many colours and, in most cases, the colours tell you about the origin of the stain.

Orange/brown stain

The problem is most often a phosphate layer (brown to light orange) on the instrument, which develops as a result of any of the following causes: water sources, detergents used to wash and clean instruments, surgical wrappings, cold sterilization solutions, or dried blood.

Black stain

The most common black stains are due to an acid reaction. Black stains may result from detergents used to clean the instrument; similar to brown stains caused by high pH in detergents. The black acid type stain can be caused by low pH (less than six) during autoclaving.

Dark brown stain

Dark brown stains are usually a result of dried blood left on an instrument. Blood should be removed from the surface of the instrument immediately. It will break down the instrument's surface with a chemical reaction.

Bluish/black stain

These are usually a result of plating and are extremely difficult to remove from the surface. The cause for this stain is the mixing of dissimilar metals in ultrasonic cleaners and during autoclaving. Multi-colour stains are most often due to excessive heat (chromium oxide stains), and actually show rainbow colours with a blue or brown overtone.

When the instrument shows these heat stains, it may have lost part of its original hardness, and may not perform well. These instruments can usually be refinished, and the hardness tested. The staining can be polished off.

Additional Staining Information for Stainless Steel Instruments

As described, stains are either deposited onto instrument's surface, plated on, or in the case of rusting, developed from the instrument itself. The most common discolouration is due to deposit stains and usually occurs during autoclaving.

To minimize staining, it is important that the autoclave runs perfectly, and that it has a well-functioning drying cycle (reminder: do not use a dry cycle on iM3 Ergo or other instruments with plastic elements). The instruments should come out completely dry, whether in wrappers or loose on a tray. If any moisture is left in the pack, or on the instruments, it will result in tiny water droplets on the instrument surface, which will leave a circular stain after drying. The colour of this stain will depend on the pH, as well as the mineral or metal contents of the water. If the drying cycle works perfectly, however, there is a much less chance for deposits to form on the surface of the instrument.

4. TECHNICAL PRODUCT DETAILS

WARRANTY INFORMATION

iM3 instruments are covered against faulty workmanship and materials. Material failure is defined as any defect, pitting, corroding or breakage due to a fault in the stainless steel. Instrument found to have these types of defects will be replaced under warranty.

The warranty does not cover instruments that break due to improper use, incorrect sterilization or cleaning, improper sharpening, or normal wear and tear (this includes instruments that have been dropped).

iM3 Ergo instruments are a registered design with the European Union 007771431-0001

TECHNICAL & SUPPORT

Call us anytime during business hours on our Tech Support Line and we'll take care of all your needs or email us with your query and we'll get back to you promptly

For servicing quotation contact us at:

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iM3 Dental Limited
iM3 Inc.
iM3 Pty Ltd.,